BAIRD
magic^{3™} INTERMITTENT CATHETER
with m³ technology

BAIRD
magic^{3™} INTERMITTENT CATHETER
with m³ technology

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BAIRD
magic^{3™} KATETER *med m³-teknologi*
TIL PERIODISK BRUK

Hydrophilic Coated Silicone
Instructions for Use
EN

Intended Use:
The catheter is intended for urinary bladder drainage in patients requiring catheterization for management of incontinence, voiding dysfunction, and surgical procedures.

Please contact your physician to determine which product options are best for you, paying close attention to product warnings/precautions and adverse reactions.

The catheter becomes slippery when wetted with water, eliminating the need for a separate lubricant. For your added convenience, this catheter is packaged with its own sterile water. Simply release the water from its foil packet and then tip the un-opened catheter package end-to-end. The catheter acts like a magnet to attract the water and activate its slippery coating. Follow these steps for best results.

1. RELEASE THE WATER

Prior to opening the sealed catheter pouch:

- Apply pressure to the foil packet to release the water.
- Ensure all water is released from the foil packet.

2. WET THE CATHETER

- Hold package with printed side up.
- Tip package end-to-end three to six times to wet catheter. This movement is required so that the water transfers back and forth over the catheter to fully wet the hydrophilic coating.

3. USE THE CATHETER

- Peel back package to expose funnel end of catheter.
- If desired, use the self-adhesive tape on the package to temporarily attach the pouch to any dry vertical surface.
- Remove catheter and use according to physician's instructions.

Warning:
This is a single use device. Do not reuse. Reuse of a single use device increases the risk of catheter acquired urinary tract infections.

Urethral catheter for urological use only.
Discard after use, do not resterilize.
Made of silicone elastomer.

Manufacturer **REF** Catalog Number

Authorized Representative **LOT** Lot Number

Rx Only Caution: Federal (USA) law restricts this device to sale by or on the order of a physician.

Single Use. **STERILE R** Sterilized using irradiation.

Not made with natural rubber latex. Do not use if package is damaged.

Not made with PVC. Use By

CAUTION: consult accompanying documents.

Manufacturer:
Rochester Medical Corporation
One Rochester Medical Drive
Stewartville, MN 55976 USA
(800) 526-4455
www.bardcare.com

Medical Product Service GmbH
Borgnasse 20
35619 Braunfels, Germany

Bard and Magic3 son marcas comerciales y/o comerciales registradas de C. R. Bard, Inc. ©2015 C. R. Bard, Inc. All Rights Reserved.

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Regulatory and identification information for Bard Magic3 catheter, including manufacturer details, registration numbers, and safety symbols.

Regulatory and identification information for Bard Magic3 catheter in Polish, including manufacturer details, registration numbers, and safety symbols.

Regulatory and identification information for Bard Magic3 catheter in Turkish, including manufacturer details, registration numbers, and safety symbols.

Regulatory and identification information for Bard Magic3 catheter in Greek, including manufacturer details, registration numbers, and safety symbols.

Regulatory and identification information for Bard Magic3 catheter in Portuguese, including manufacturer details, registration numbers, and safety symbols.

Regulatory and identification information for Bard Magic3 catheter in Romanian, including manufacturer details, registration numbers, and safety symbols.

Regulatory and identification information for Bard Magic3 catheter in Czech, including manufacturer details, registration numbers, and safety symbols.

Regulatory and identification information for Bard Magic3 catheter in Hungarian, including manufacturer details, registration numbers, and safety symbols.

Regulatory and identification information for Bard Magic3 catheter in Slovak, including manufacturer details, registration numbers, and safety symbols.

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Bard and Magic3 are trademarks and/or registered trademarks of C. R. Bard, Inc. ©2015 C. R. Bard, Inc. Alle rettigheder forbeholdes.