Antibacterial Coated Silicone, Silicóna con revestimiento antibacterial, Revêtu de silicone antibactérien, Antibakteriell beschichtetes Silikon, Silicone antibatterico rivestito, Antibakteriell Belag Silikone, Antibacterieel silicóne met deklaag, Antibakteerinen silikóniva, Antibakterieel Silikon, Til PERIODISK BRUK

Contains Nitrofurazone, Contiene nitrofurazona, Contient de la nitrofurazone, Enthält Nitrofurazon, Contiene nitrofurazona, Innehåller nitrofurazon, Sisältää nitrofurazonia, KÄYTTÖOHJEET, Til PERIODISK BRUK

INTENDED USE, The catheter is intended for urinary bladder drainage in adult males and females requiring catheterization for management of incontinence, voiding dysfunction, and surgical procedures. Efficacy of the catheter in preventing urinary tract infection during intermittent use has not been established. The device is not intended to be used as a treatment for active urinary tract infection.

Please contact your physician to determine which product options are best for you, paying close attention to product warnings/precautions and adverse reactions.

ACTIVE INGREDIENT 10.2 +/- 2.0 micrograms Nitrofurazone (5 nitro-2-furaldehyde semicarbazone) per mm² catheter. INGREDIENTE ACTIVO 10, 2 +/- 2,0 microgramos de nitrofurazona (5 nitro-2-furaldehido semicarbazona) por mm² de cateter. Katheter-Gesamtoberfläche (mm²), Surface totale de la sonde (mm²), Superficie utile totale del cateter (mm²)

Table with columns: Fr, Male Length, Female Length, Surface area (Fr, Male Length, Female Length). Includes values for lengths (12, 14, 16, 18, 20) and surface areas.

The silicone layer on the external surface of the catheter is impregnated with nitrofurazone. Nitrofurazone is a broad-spectrum antimicrobial boundary producing local antibacterial activity. Nitrofurazone is not significantly absorbed through the urethra for the period of 12-24 hours.

WARNING: This is a single use device. Do not reuse. Reuse of a single use device increases the risk of catheter acquired urinary tract infection. CONTRAINDICATIONS: Do not use in individuals with known sensitivity to nitrofurazone.

PRECAUTIONS: The following adverse events may be associated with the use of urethral catheters: infections, paraphimosis, urethral tear or creation of false passages, urethral strictures, meatal stenosis, urethral damage, bleeding, bladder spasms, bladder damage, catheter blockage, and leakage.

Use of certain antimicrobial agents may allow overgrowth of non-susceptible organisms including yeast and fungi. If this occurs, or if irritation, sensitization or superinfection develops, discontinue use and institute appropriate therapy.

Pediatric use: Safety and effectiveness in children has not been established. Pregnancy: Pregnant women should consult their physician before using this product.

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PRECAUTIONS: Avoid the use of drug insertion devices in the urethra of pregnant women. Avoid the use of drug insertion devices in the urethra of pregnant women.

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