(EN)

Non-adhesive, All-Silicone Male External Catheter with Strap

DESCRIPTION/INDICACION
The NATURAL™ Catheter is a non-adhesive, all-silicone male external catheter designed for the management of male urinary incontinence.

CONTRAINDICATION
Do not use on irritated or compromised skin.

PRECAUTIONS
Do not use if allergic reaction occurs. For good hygiene, change catheter daily. Use of a single device for periods longer than 24 hours may increase the risk of complications. Applying strap tightly may cause injury to the penis from decreased circulation. Keep strap clean and dry.

DIRECTIONS: To Apply Catheter

- Wash penis with mild soap and warm water. Dry thoroughly.
- Trim pubic hair if necessary.
- Unroll catheter over penis. 
- Gently wrap strap over catheter around penis and secure with hook and loop closure. Ensure strap remains over catheter sheath. (To maintain hook and loop closure function, keep hook and loop closed when strap is not in use.) 
- IMPORTANT:** To avoid injury, do not overtighten strap.
- Connect to drainage device.

TO REMOVE CATHETER

- Remove strap by separating hook and loop closure. 
- Gently roll catheter off the penis. 

Manufacturer:
Rochester Medical Corporation
One Rochester Medical Drive
Stewartville, MN 55976 USA
(800) 526-4455 www.bardcare.com

EC REP Medical Product Service GmbH
Bongasse 20
35619 Braunsfels, Germany

Bard and Natural are trademarks and/or registered trademarks of C. R. Bard, Inc. ©2016 C. R. Bard, Inc. All Rights Reserved.

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