

# URISLEEVE® Leg Bag Holder

## BARD® Comprehensive Care

Supporting your Collection System


Easy to use, URISLEEVE® Leg Bag Holder provides total security and maximum comfort as an alternative to leg straps.

The soft fabric of URISLEEVE® gives support, applying an even pressure over the whole area of the URIPLAN® Leg Bag, holding it secure.





URISLEEVE® is available in four sizes and easily adapts to fit on the calf or on the thigh. To determine the right size of URISLEEVE® for you, measure the circumference of either:

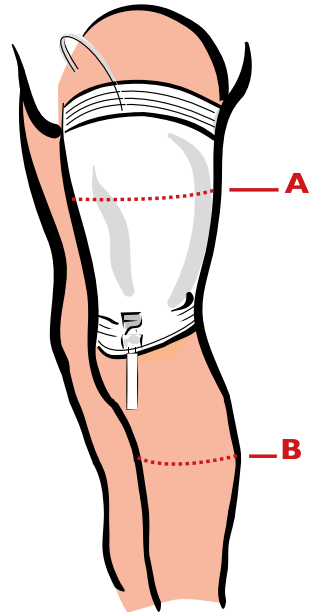- A)** the mid-point of your thigh, or
- B)** the widest point of your calf.

The fitting guide below will indicate which product you need. URISLEEVE® is available on prescription from your Nurse or Doctor.



### Fitting Guide

URISLEEVE® Size	Colour Code	Circumference	Product Code
Small	Yellow 	24-39 cm	150111 <input type="checkbox"/>
Medium	Blue 	36-55 cm	150121 <input type="checkbox"/>
Large	Brown 	40-70 cm	150131 <input type="checkbox"/>
Extra Large	Green 	65-95 cm	150141 <input type="checkbox"/>

Size recommendations are to be used as a guide only


# URISLEEVE®

LEG BAG HOLDER

## Fitting Instructions

Simply pull the URISLEEVE® Leg Bag Holder over the foot like a sock and up onto the thigh or calf with the coloured stripe at the top and facing outwards.

Slide your BARD® URIPLAN® Leg Bag into the top of the URISLEEVE® Leg Bag Holder through the slit in the material and push the drainage tap through the hole at the bottom.

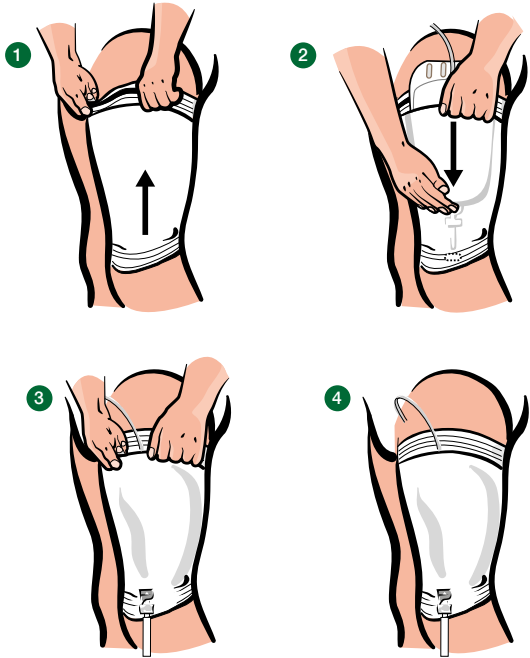

## Washing Instructions



The URISLEEVE® Leg Bag Holder is made of 80% Polyester / 20% Elastane and is machine washable.



**FREEPHONE 0800 0121 699**

**Fax: 01903 875 085**

**Email: [ukcustomercare@crbard.com](mailto:ukcustomercare@crbard.com)**

**Web: [www.scripteasy.co.uk](http://www.scripteasy.co.uk)**



TYTEX A/S, Industrivej 21, DK-7430 Ikast

Distributed by Bard Limited, Forest House, Tilgate Forest Business Park, Brighton Road, Crawley, West Sussex, RH11 9BP.  
Tel: +44 (0) 1293 527 888 Fax: +44 (0) 1293 552 428 Bard Customer Care +44 (0) 1293 529 555

Bard, Uriplan and Urisleeve are registered trademarks of C. R. Bard, Inc., or an affiliate. © 2014, C. R. Bard, Inc.  
All rights reserved. Please Consult product labels and inserts for any indications, contraindications, hazards, warnings cautions and instructions for use. 0814/4081

**NURSE ADVICE LINE FREEPHONE 0808 168 4048**

**[www.bardmedical.co.uk](http://www.bardmedical.co.uk)**

**BARD | MEDICAL**