
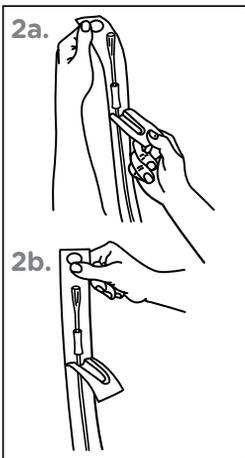
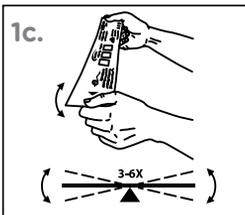


# HOW TO PREPARE YOUR MAGIC<sup>3</sup><sup>®</sup> HYDROPHILIC INTERMITTENT CATHETER

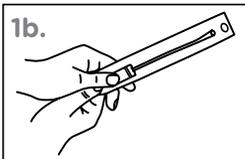

## Step 1

### *Prior to Opening the Package*

- Wash your Hands Thoroughly with Soap and Water**
- Break the Water Packet** by applying pressure to the foil packet to release the water (Hint: fold the water packet prior to applying pressure to ease release). Ensure all water is released from the foil packet.
- Wet the Catheter** by holding the package with the printed side up and tipping the package end-to-end three to six times. This movement is required so that the water transfers back and forth over the catheter to fully wet the hydrophilic coating.

## Step 2

### *Opening the Package*

- Peel Open the Package** at the funnel end to expose approximately 4" of the catheter. Do not remove the catheter yet.
- For convenience use the adhesive tab at the funnel end of the package to **stick the package to a nearby dry vertical surface** while preparing to catheterize.

*Proceed with catheterization. See "How to Catheterize Using Your MAGIC<sup>3</sup><sup>®</sup> Hydrophilic Intermittent Catheter" on reverse side.*

Summary of Indications & Warnings: Intermittent catheters are intended for use by adult and pediatric, male and female patients for draining urine from the bladder. Reuse of a single use device may create a risk of patient or user infection; compromise the device which may lead to device failure and/or injury, illness or death of the patient. Please consult product labels and inserts for more detailed safety information and instructions for use.

**BARD**

**CARE**

Care. Comfort. Choice.

www.bardcare.com  
Phone: 800 243 3315  
Fax: 800 859 5205

Bard, Magic3 and Sure-Grip are trademarks and/or registered trademarks of C. R. Bard, Inc.  
©2015 C. R. Bard, Inc. All Rights Reserved.  
1502-25 R02/15 THP P04/15 7.5M

# HOW TO CATHETERIZE USING YOUR MAGIC<sup>3</sup><sup>®</sup> HYDROPHILIC INTERMITTENT CATHETER

*Always try to pass urine and empty your bladder as much as you can before using an intermittent catheter.*

## Steps 1 and 2

### **Prepare Your MAGIC<sup>3</sup><sup>®</sup> Hydrophilic Catheter for Use.**

See reverse side for instructions.

## Step 3

**Wash Meatus.** Thoroughly wash area around meatus before catheterizing.

## Step 4

**Wash your Hands Thoroughly.**

## Step 5

### **Using the Catheter**

#### *With SURE-GRIP™ Sleeve*

- Hold the SURE-GRIP™ Sleeve with your dominant hand and squeeze it to hold the catheter shaft as you remove the catheter from the pack.
- Next, hold the catheter funnel above the SURE-GRIP™ Sleeve with your other hand and slide the SURE-GRIP™ Sleeve down the shaft, stopping at around 6" from the tip. Release the funnel.
- While gripping the catheter with the SURE-GRIP™ Sleeve, gently pass the tip of the catheter into your urethra until the SURE-GRIP™ Sleeve nears the meatus. If there is no urine flow, loosen the tension on the SURE-GRIP™ Sleeve to allow it to slide back towards the funnel end. Re-grip the SURE-GRIP™ Sleeve and continue to insert the catheter into the urethra. Repeat until urine starts to flow.

#### *Without SURE-GRIP™ Sleeve*

- Hold the funnel end and remove the catheter from the packaging.
- Gently pass the tip of the catheter into your urethra and advance the catheter until urine starts to flow.

## Step 6

**Drain Urine.** As soon as urine starts to flow, point the catheter towards the toilet or other collection device. Try to keep the catheter steady until urine stops flowing.

## Step 7

**Withdraw Catheter.** When urine stops flowing, slowly withdrawal the catheter, stopping if flow starts again, until the last few drops have drained.

## Step 8

**Dispose Catheter and Packaging.** Finish by disposing of the catheter and its packaging. Wash your hands with soap and water.