

# Urisleeve®

## LEG BAG HOLDER

### Supporting your Collection System


Easy to use, URISLEEVE® Leg Bag Holder provides total security and maximum comfort as an alternative to leg straps.

The soft fabric of URISLEEVE® gives support, applying an even pressure over the whole area of the URIPLAN® Leg Bag, holding it secure.





URISLEEVE® is available in four sizes and easily adapts to fit on the calf or on the thigh. To determine the right size of URISLEEVE® for you, measure the circumference of either:

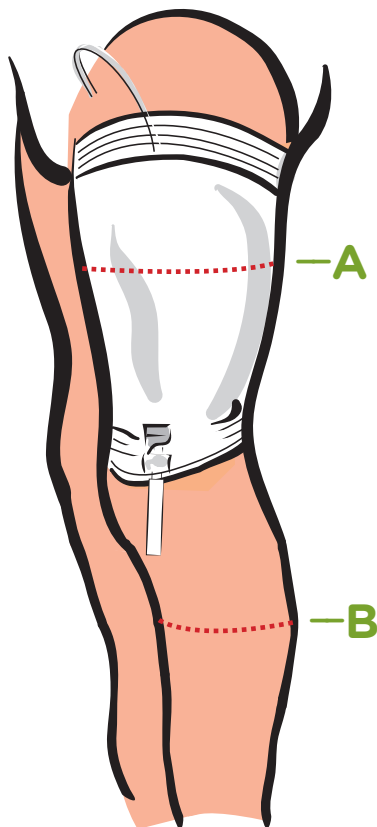- A) the mid-point of your thigh, or
- B) the widest point of your calf.

The fitting guide below will indicate which product you need. URISLEEVE® is available on prescription from your Nurse or Doctor.



### Fitting Guide

URISLEEVE® Size	Colour Code	Circumference	Product Code	
Small	Yellow	 24-39 cm	150111	<input type="checkbox"/>
Medium	Blue	 36-55 cm	150121	<input type="checkbox"/>
Large	Brown	 40-70 cm	150131	<input type="checkbox"/>
Extra Large	Green	 65-95 cm	150141	<input type="checkbox"/>

Size recommendations are to be used as a guide only


# Urisleeve®

## LEG BAG HOLDER

### Fitting Instructions

Simply pull the URISLEEVE® Leg Bag Holder over the foot like a sock and up onto the thigh or calf with the coloured stripe at the top and facing outwards.

Slide your BARD® URIPLAN® Leg Bag into the top of the URISLEEVE® Leg Bag Holder through the slit in the material and push the drainage tap through the hole at the bottom.

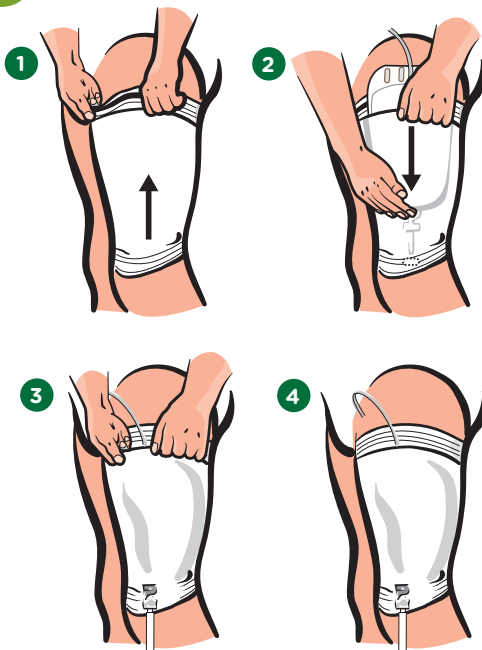

### Washing Instructions



The URISLEEVE® Leg Bag Holder is made of 80% Polyester / 20% Elastane and is machine washable.



**FREEPHONE 0800 0121 699**

**Fax: 01903 875 085 Email: [ukcustomercare@crbard.com](mailto:ukcustomercare@crbard.com) Web: [www.scripteasy.co.uk](http://www.scripteasy.co.uk)**

TYTEX A/S, Industrivej 21, DK-7430 Ikast

Distributed by Bard Limited, Forest House, Tilgate Forest Business Park, Brighton Road, Crawley, West Sussex, RH11 9BP.  
Tel: +44 (0) 1293 527 888 Fax: +44 (0) 1293 552 428 Bard Customer Care +44 (0) 1293 529 555

Bard, Script-easy, Uriplan and Urisleeve are registered trademarks of C. R. Bard, Inc., or an affiliate. © 2016, C. R. Bard, Inc. All rights reserved. Please consult product labels and inserts for any indications, contraindications, hazards, warnings cautions and instructions for use. 0316/4822 15-013

**NURSE ADVICE LINE FREEPHONE 0808 168 4048**

**BARD**

**CARE**

Care. Comfort. Choice.

[www.bardcare.uk](http://www.bardcare.uk)